

INSTRUCTIONS FOR USE
Aluminium 4-wheeled rollator
Product code: TGR-R RA 885



1. Characteristics


Adjustable 4-wheeled rollator is made of aluminium frame which is coated with paint with metallic lustre. The rollator is equipped with brakes with locking mechanism, upholstered and height adjustable seat, backrest, and shopping bag. The product is supplied with solid wheels with narrow cross section 8'' in diameter which can be dismantled for transport, and height adjustable handles with anatomic handgrips.

The set includes:

- Frame with seat
- Backrest
- Shopping bag with stiffening
- Two push handles with rear wheels
- Front wheels
- Locking knobs – 2 pcs
- Locking pin - 2 pcs

Elements of the rollator:

- a.** push handle
- b.** brake
- c.** locking knob (push handles)
- d.** locking clip (backrest)
- e.** backrest
- f.** seat
- g.** shopping bag
- h.** locking pin (rear wheels)

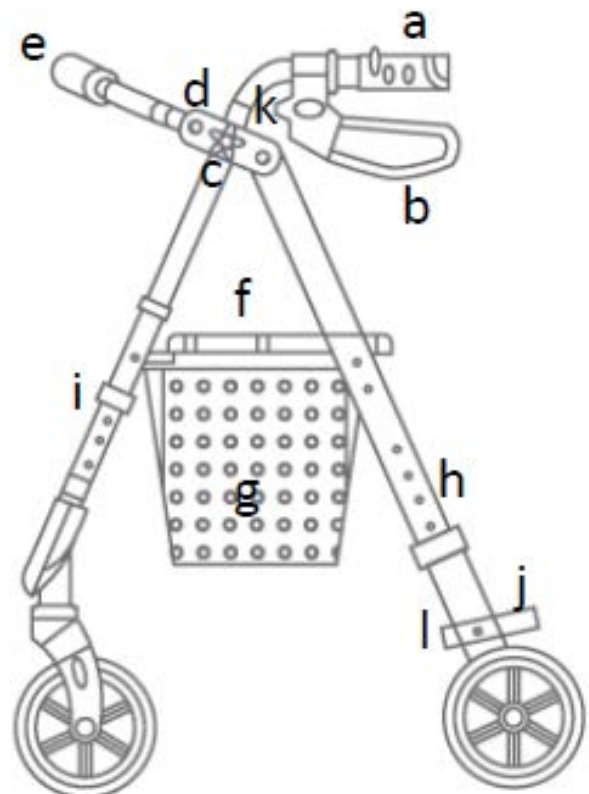

- i. locking clip (front wheels)
- j. cane holder
- k. upper adjustment of brake
- l. lower adjustment of brake

2. Application

The rollator is designed for patients with reduced mobility who need stable support and help while walking. It is also recommended to the elderly as an aid in keeping balance, improving walking comfort and ensuring safety from falling. The use of the rollator affects positively on the return to activity of the people with locomotion and tilting to erect position problems. It guarantees better stabilization, support and security while walking.

3. Assembly

Unfolding and folding the rollator

- To unfold the rollator, pull the frame apart and press the seat until the rollator is completely unfolded.
- Rear wheels and push handles are joined by the brake cables, thus firstly, fix push handles into the mounting openings and blocked them with locking knobs. Then, block the rear wheels with pins.
- Fix front wheels in the front part of the frame and secure them with locking clips which are placed in the above height adjustment openings.
- Fix backrest in the upper part of the frame and block it with locking clips.
- Remove plastic casing from wheels.
- To fold the rollator, lift up the seat and the bar which is placed under the seat.

NOTE! Be careful while folding or dismantling the rollator due to the risk of smashing fingers in hinges.

Brakes

- To stop the rollator, pull the brake handle upwards and hold.
- While stop or using the seat, lock the brake with locking mechanism by pushing down the brake handle.
- To release the brake, pull the brake handle upwards.

Height adjustment of handles

- To adjust the height of the handles, unscrew the locking knob which is located on the outer side of the frame, select appropriate height and tighten the locking knob.

Note! The handles must be set on the same level.

Height adjustment of seat

- To adjust the height of the seat, remove the locking clip (for front wheels assembly) and press the locking pin (for rear wheels assembly). Select appropriate height and block the wheels with locking clip and pin.

Adjustment of brakes

Adjustment of upper brake

- Unscrew the nut of the cable, pull it up to the appropriate length and lock it by tightening the locking nut.

Adjustment of lower brake

- Loosen the mounting screw with the use of open-end wrench, pull the black brake cable to adjust the tension of the cable, and then, tighten the mounting screw.

4. Technical data

Overall width	65 cm
Overall depth	69 cm
Seat dimensions	40 cm x 25.5 cm
Seat height from the ground	63 cm

Handles height adjustment	90-100 cm
Weight	6.8 kg
Maximum capacity	130 kg
Maximum capacity of the shopping bag	5 kg
Frame colour	graphite

5. Notes

Timago International Group does not bear responsibility of improper use of the rollator, failure to comply safety regulations and misuse.

6. Cleaning and maintenance

- Clean the frame with damp cloth and dry thoroughly.
- Do not use strong polishing agents due to the damage of protecting coatings.
- Clean the dirt from moving parts of the rollator regularly (hinges, clamping screws, locks).

7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date

	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors.
Learn more at: www.timago.com. Thank you for choosing Timago!



Timago International Group
Spółka z o.o. i Spółka – Spółka komandytowa
ul. Karpacka 24/12
43-316 Bielsko-Biała, Polska

T.: +48 33 499 50 00
F.: +48 33 499 50 11
E.: info@timago.com



Rev. II, date 20.05.2021